



**Advanced Vacuum Cleaning Solution** is designed to remove debris from vacuum system outlets and all subsidiary vacuum components. After time and heavy use, vacuum outlets can become restricted by the buildup of debris. These buildups can be easily removed by inserting a small amount of **Advanced Vacuum Cleaning Solution** into the outlet. This process may eliminate the need to isolate, disassemble and rebuild the outlet.

**Advanced Vacuum Cleaning Solution** is designed to increase the restricted vacuum flow of sluggish medical vacuum system outlets, adapters, hose assemblies, etc. **Advanced Vacuum Cleaning Solution** is easy, ready-to-use and designed to work on partially plugged outlets/equipment.

**ADVANCED VACUUM CLEANING SOLUTION may be used for all Vacuum System applications, including:**

- ◆ Operating rooms
- ◆ Patient rooms
- ◆ Vacuum regulators
- ◆ Vacuum adapters
- ◆ Recovery rooms
- ◆ Labor & delivery rooms
- ◆ Portable vacuum units
- ◆ Vacuum hose assemblies

## Instructions for Use:

**Advanced Vacuum Cleaning Solution** is safe, easy, ready-to-use and is designed to work on partially plugged outlets/equipment.

1. Use an appropriate adapter and tubing (*small I.D. tube is recommended*)
2. Suction approximately 1 cup of cleaning solution from the container, applying to each outlet.
3. Let stand for approximately 15 minutes to allow the solution to soften any existing debris.
4. After the solution has softened debris, suction up approximately 1 cup of warm water and apply to each outlet, flushing away cleaning solution.
5. Repeat if necessary.
6. If problem persists or the outlet/line is completely plugged, disassembly and repair may be required.

**Warning!!** Follow the above steps carefully. **Never** apply large amounts of liquid. The liquid can be pulled back into the vacuum pumps. Large amounts of liquid could damage vacuum source equipment.

**NEVER** use any flammable product, such as alcohol, to flush vacuum equipment.



\*A case contains 4 bottles with an approx. shipping weight of 36lbs



**CAUTION:** When handling any chemical solutions, avoid direct contact with skin. Use rubber gloves and protective eye wear.

KEEP OUT OF THE REACH OF CHILDREN

## Advanced Vacuum Cleaning Solution:

- ◆ Clears sluggish medical vacuum system outlets.
- ◆ Increases restricted vacuum flow.
- ◆ Cleans vacuum system components and accessories.
- ◆ Deodorizes.
- ◆ Easy to use.
- ◆ Eliminates costly downtime.
- ◆ Safe - when used as directed - for all vacuum components.
- ◆ When used as instructed Advanced Vacuum Cleaning Solution is harmless to vacuum pumps.

**airandvac.com**

e-mail to client

[www.airandvac.com](http://www.airandvac.com)

1954 Friendship Drive, Suite 103  
El Cajon, CA 92020

Phone: 619.562.3060 or 800.743.8280  
Fax: 619.562.3145

E-mail: [support@airandvac.com](mailto:support@airandvac.com)

FORM # 5100REV0711